
Induktionsschlinge EZT 3011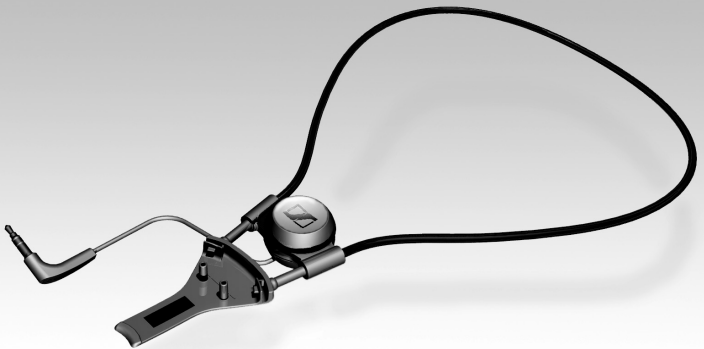

Bedienungsanleitung | Instruction manual
Notice d'emploi | Istruzioni per l'uso
Instrucciones de uso | Gebruiksaanwijzing

Deutsch

English

Français

Italiano

Español

Nederlands

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Important safety instructions

- Please read this instruction manual carefully and completely before using the device.
- Make this instruction manual easily accessible to all users at all times. Always include this instruction manual when passing the device on to third parties.
- Do not use the device in situations which require special attention (e.g. when performing skilled jobs).
- Always keep the device dry and do not expose them to extreme temperatures.

Intended use of the device

Intended use includes:

- having read this instruction manual, especially the chapter “Important safety instructions”,
- using the device within the operating conditions and limitations described in this instruction manual.

“Improper use” means using the device other than as described in this instructions, or under operating conditions which differ from those described herein.

The EZT 3011 induction loop

The EZT 3011 induction loop is an accessory for the infra-red receiver of the Set 830 S TV listening system and for the RF receiver of the Set 840 S TV listening system. The EZT 3011 inductively couples the audio signal from the receiver to hearing aids.

Using the volume control on the receiver the sound may be easily adjusted to suit an individual user's hearing. This allows hearing aid wearers to take advantage of an interference free, optimum sound quality, wireless, listening experience.


Delivery includes

- 1 EZT 3011 induction loop
- 1 instruction manual


Putting the EZT 3011 into operation

Connecting the induction loop to the receiver

- ▶ Attach the clip of the induction loop to the receiver as shown.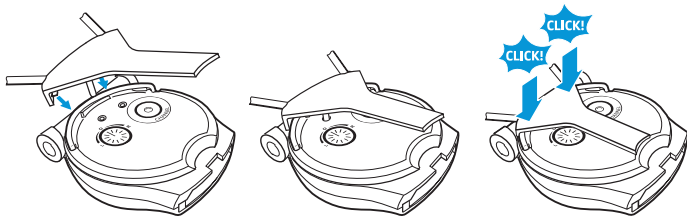

- ▶ Connect the 3.5 mm jack plug to the socket on the receiver.

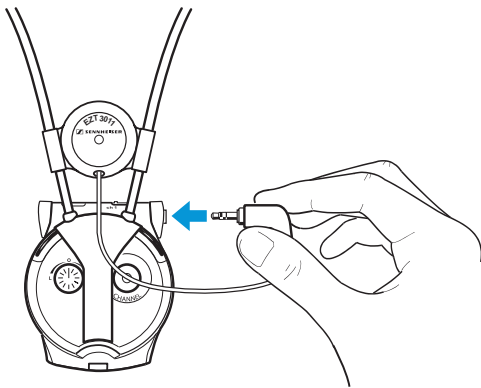- ▶ Set your hearing aid to the T-position. If your hearing aid has no T-position, please contact your hearing aid acoustician.

For alternative connection possibilities to your hearing aid as well as the connection to cochlea implants, please contact your hearing aid acoustician.

CAUTION! Hearing damage due to high volumes!


Listening at high volume levels for long periods can lead to permanent hearing defects.

- ▶ Before using the receiver, set the volume to a low value.
 - ▶ Do **not** continuously expose yourself to high volumes.
-

Wearing the receiver around the neck

- ▶ Place the induction loop around your neck, with the Sennheiser logo facing away from your body.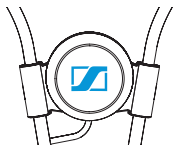

- ▶ Switch your audio source on.
- ▶ Switch the receiver on and enjoy the excellent sound quality.

For further information on the operation of your receiver, please refer to the instruction manual of the receiver.

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- ▶ Use a slightly damp cloth to clean the induction loop.

Specifications

EZT 3011 induction loop

Audio connector	3.5 mm stereo jack plug
Cable length	75 cm
Impedance	28 Ω at 1 kHz

Manufacturer Declarations

Warranty

Sennheiser electronic GmbH & Co. KG gives a warranty of 24 months on this product. For the current warranty conditions, please visit our web site at www.sennheiser.com or contact your Sennheiser partner.

WEEE Declaration

Please dispose of this product at the end of its operational lifetime by taking it to your local collection point or recycling centre for such equipment.

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